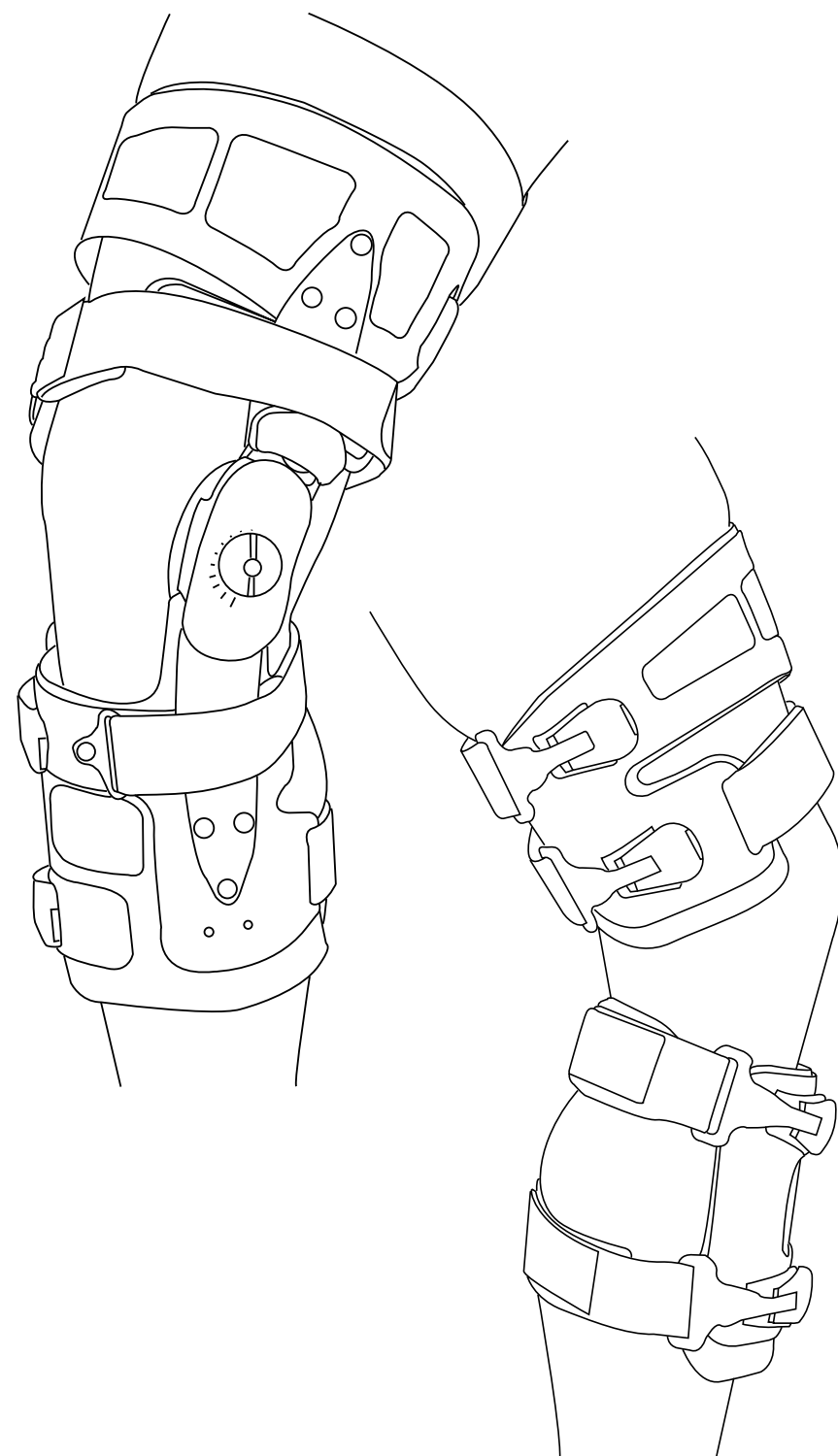
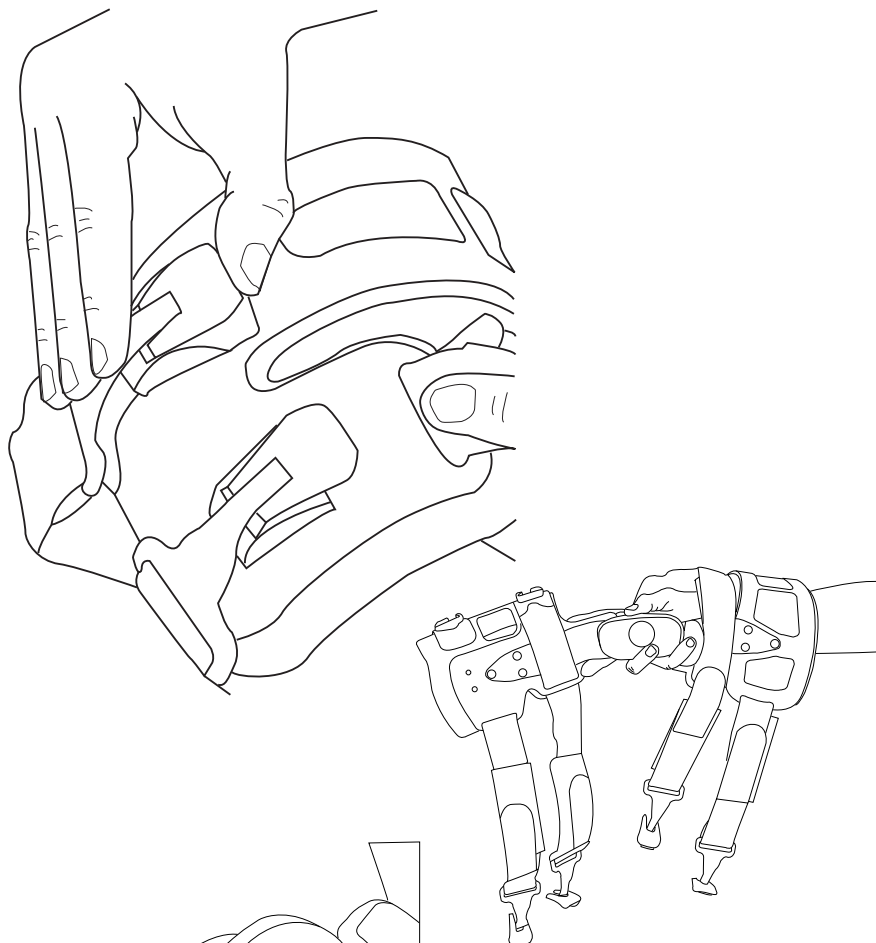
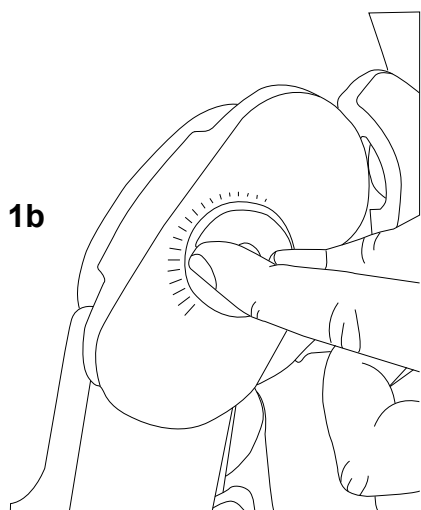
Thruster RLF

Application Instructions CP020218 Rev E 03/08

MUSCLE-POWERED, FUNCTIONAL KNEE BRACE FOR OSTEOARTHRITIS

1a**1b****1. Unfasten Buckle**

- a.** Unfasten all of the buckle ends by pushing the gray button on each buckle.
- b.** If this is the first time the brace is applied, the brace should be at its lowest setting. If not, turn the force knob to its lowest setting.

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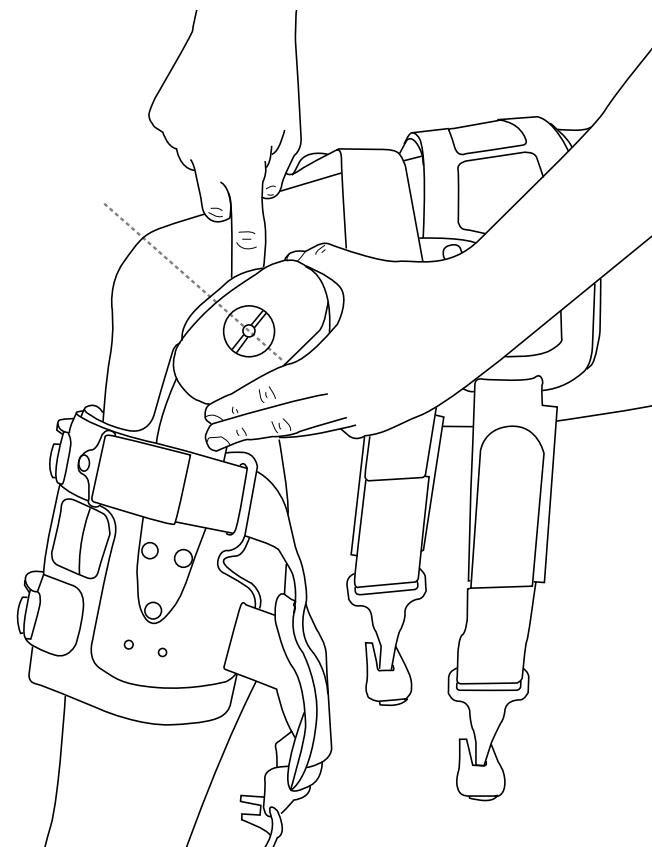
2**2. Apply the Brace**

Apply the brace with the patient in a seated position and the leg bent at 80°-90°.
Medial (MA) Brace: Place the hinge on the inside of the leg slightly higher and behind the joint axis (the adductor tubercle, or large bony knot on the inside of the leg).
Lateral (LA) Brace: Place the hinge on the outside of the leg slightly higher and behind the joint axis.

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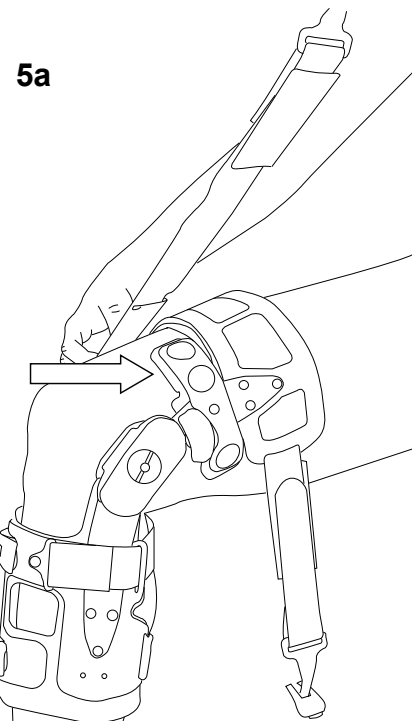
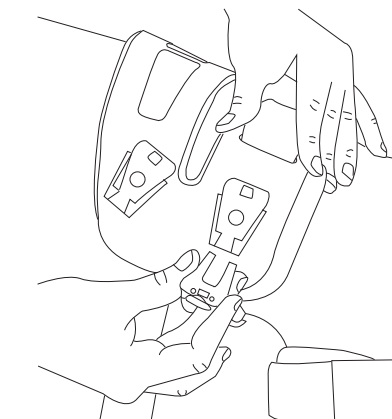
5a**5b****5. Fasten Lower Thigh Strap**

- a.** Wrap the lower thigh strap firmly around the back of the leg and engage buckle on the opposite side of the shell.
- b.** Adjust the length of the strap using the fastener at the buckle end of the strap.

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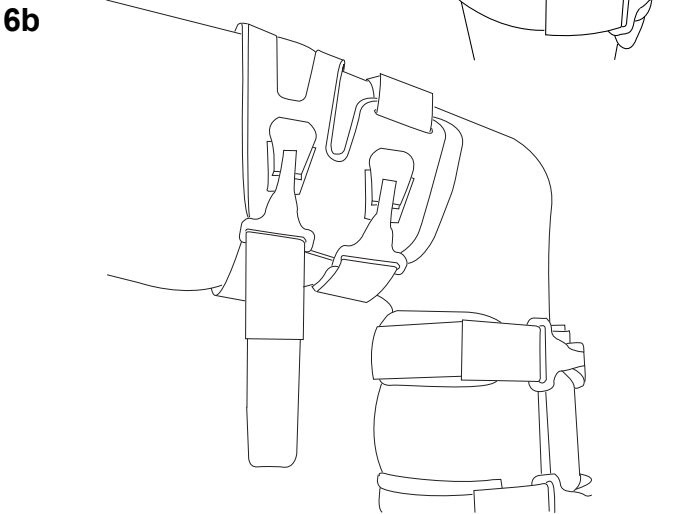
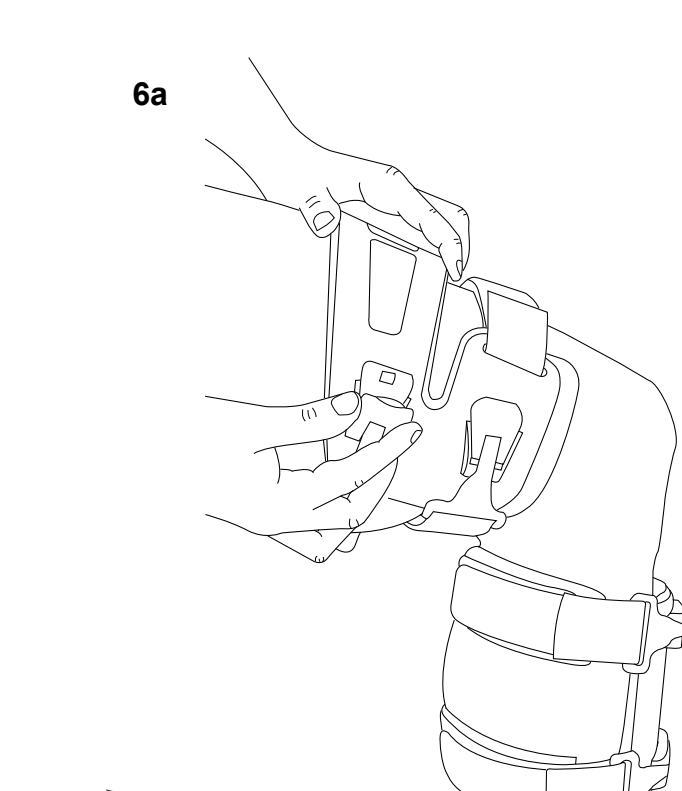
- 6. Fasten Upper Thigh Strap**
a. Wrap the upper thigh strap around and behind the leg to engage the buckle.
b. Adjust the strap using the fastener at the buckle end of the strap.
c. If necessary, stand and tighten straps.

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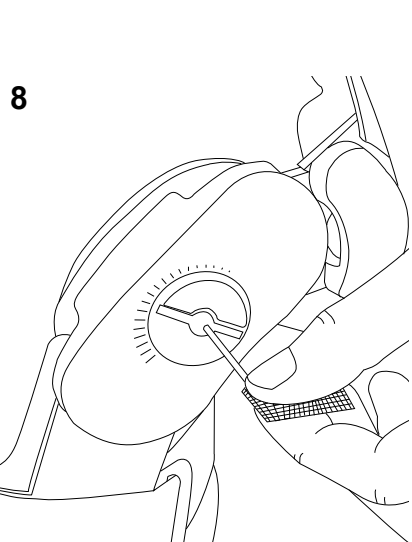
7. Hinge Adjustment
 Stand and walk in the brace to see how it feels on the leg. Some patients may experience pain relief prior to adjusting the hinge, due to varying leg angulations. The brace only applies force as it approaches full extension. It can only be adjusted while seated with the leg bent at 90°. If more force is required, sit down and adjust the dial to an increased force setting. Walk again to test for pain relief.

NOTE: Only adjust the hinge enough for pain relief. Walk after each adjustment. Too much force may cause discomfort and excessive muscle soreness. Once you reach pain relief, don't adjust the hinge any further. It takes at least a week for the leg to get used to having pressure applied to it. If there is any pinching from the straps, bend the leg to 90° and readjust the straps for comfort. It is important that the straps closest to the knee are snug, or you will lose brace effectiveness.

8. Locking the Hinge (Optional)
 To guard against accidental movement of the force dial, use the hex wrench (provided) to lightly tighten the screw.

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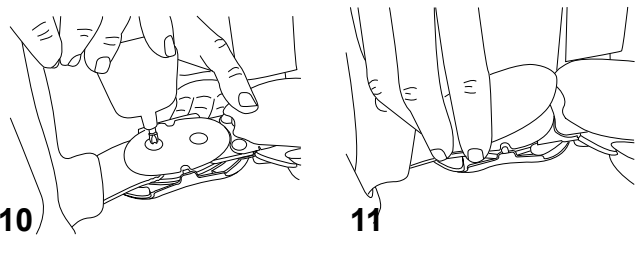
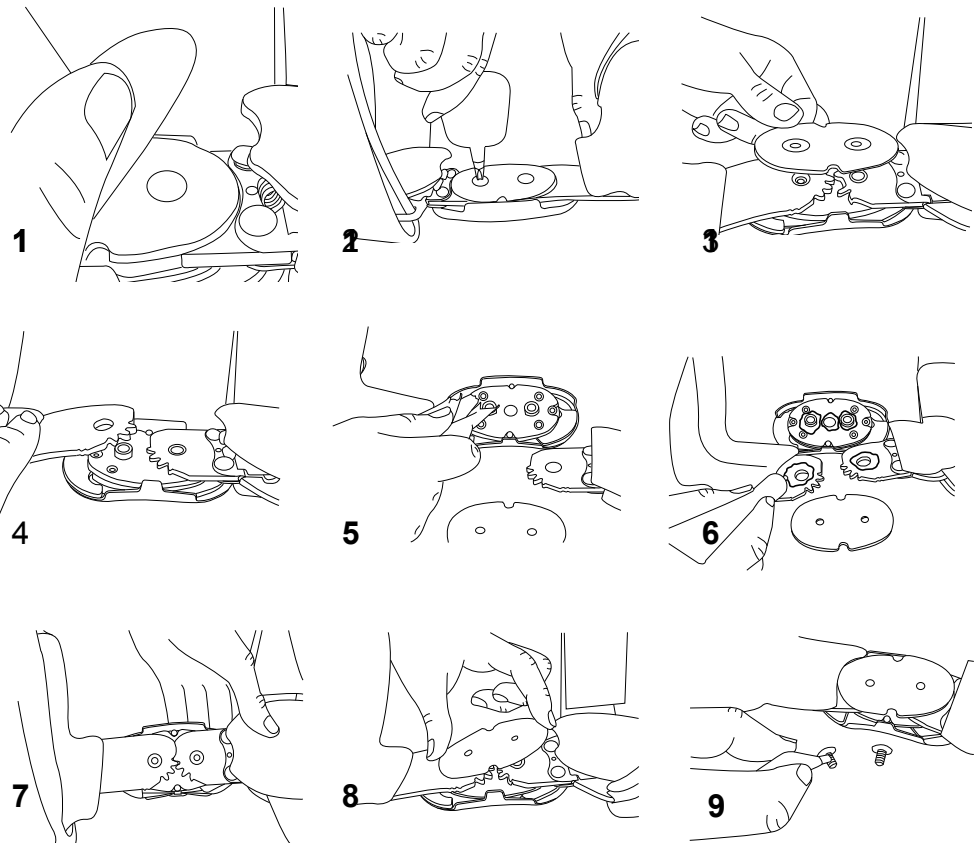
Hinge Care

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient. This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.

Indications: Thruster RLF MA is indicated for Medial Unicompartmental Gonarthrosis. Thruster RLF LA is indicated for Lateral Unicompartmental Gonarthrosis. Each brace may help to relieve symptoms of pain and inflammation. These braces are designed to aid in delaying osteotomy or total joint procedures, to increase daily walking time while increasing activities, and to aid in slowing the progression of the condition. These braces are also indicated for post-operative or post-injury application following collateral ligament injury or reconstruction (when worn on the opposite side from the injury) to relieve stress on these healing structures and post-meniscal repairs.

Contraindications: Thruster RLF MA is contraindicated for Lateral Unicompartmental Osteoarthritis. Thruster RLF LA is contraindicated for Medial Unicompartmental Osteoarthritis. Both are contraindicated

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For product information or questions pertaining to sales or service, please contact the national distributor in your area or Bledsoe Brace Systems directly.

Manufactured by:
Bledsoe Brace Systems™
 2601 Pinewood Drive, Grand Prairie, Texas, 75051, USA
 Toll Free Tel.1.888.BLED SOE (253.3763)
 Or 1.800.527.3666 • Local 972.647.0884
 Local FAX 972.660.5495
 International FAX 972.606.0649
www.BledsoeBrace.com

for tri- or bi-compartmental osteoarthritis, or for knees with injury or chronic stretch of the collateral ligament on the same side of the leg as the brace placement.

WARNING: Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.

Warranty: This device is warranted to be free from defects in material and workmanship for a period of 90 days for wraps, pads and straps and normal wear components; 12 months on all other parts; and 3 years against hinge breakage. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.BledsoeBrace.com, or by calling 1.800.527.3666.

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