


## Clinic Shoulder Sling Immobilizer


### APPLICATION INSTRUCTIONS

CP020424 Rev A 08/08


This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

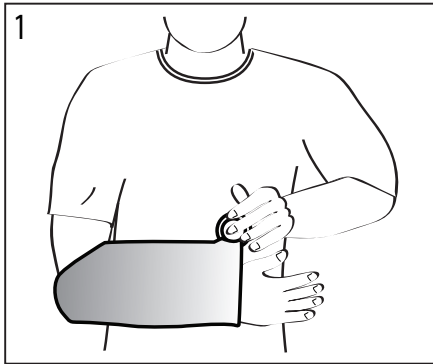


1. Place arm in sling.

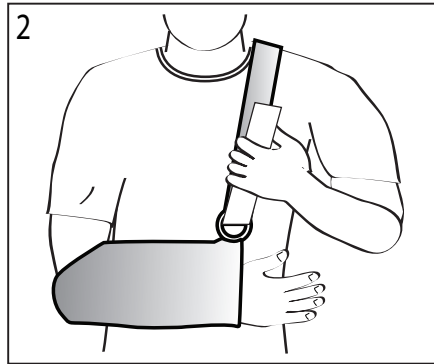

2. Secure Shoulder Strap  
Pull shoulder strap through the rings and secure using the fastener.

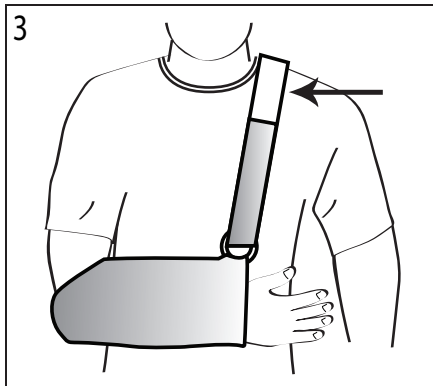

3. Adjust Shoulder Pad  
Adjust the shoulder pad to fit around patient's neck.

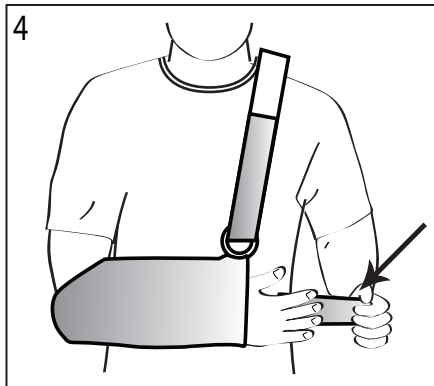

3. Secure Waist Band  
Wrap the waist band around the patient. Pull through the ring and secure using fastener.

**INDICATIONS:** Indicated for support of the upper extremity.

**CONTRAINDICATIONS:** Contraindicated for unstable fractures of the upper extremity.

**WARNINGS:** Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time. If any additional pain or symptoms occur while using this brace, discontinue use and seek medical attention.

**WARRANTY:** This device is warranted to be free from defects in material and workmanship for a period of 90 days. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at [www.BledsoeBrace.com](http://www.BledsoeBrace.com), or by calling 1.888.BLEDSOE (253.3763).



Council Directive 93/42/EEC  
of 14 June 1993 concerning Medical devices

MANUFACTURED BY:  
**BLEDSOE BRACE SYSTEMS™**  
2601 Pinewood; Grand Prairie, Texas, 75051, USA  
Toll Free Tel: 1.888.BLEDSOE (253.3763) • 1.800.527.3666  
Local: 972.647.0884 • Local Fax: 972.660.5495  
International Fax: 972.606.0649  
[www.BledsoeBrace.com](http://www.BledsoeBrace.com)

EUROPEAN AUTHORIZED REPRESENTATIVE

EC REP

EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands  
Tel: (+31) 70 345 8570  
Fax: (+31) 70 346 7299