



**Bledsoe
Brace
Systems™**

Bledsoe Extender Arm Brace



**Alternative to
casting or
splinting with
range of motion**



Original Arm Brace

- **Easy hinge adjustment:** Hinge is quick setting and limits extension from 0° to 90°, flexion from 0° to 135°, and locks 0° to 90° in 15° increments.
- **Reformable:** Formable aluminum plates adjust to fit virtually any arm circumference.
- **Reapplication:** After initial fitting, the brace can be easily removed in one piece and reapplied by the patient with one hand.
- **Patient comfort:** Open, light weight design for patient comfort. The lateral hinge and arm design prevents rubbing against the body or abrasions.
- **Increased stability:** Detachable, formable wrist-arm splint for increased stability of the forearm-wrist-hand.
- **Fast, easy fitting and unique push-button adjustment - no tools required:** In most cases, the Bledsoe Extender Arm can be formed, fitted and adjusted in 5 minutes or less.
- **Bledsoe Extender Arm Brace** is designed to handle post-op elbow arthroscopy, severe elbow strains and ligament trauma. The Bledsoe Extender Arm Brace should be used when limited R.O.M. or locked control of the elbow is desired.

Bledsoe Extender Arm Brace

Restrictions on use

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

Warning

Do not leave this device in the trunk of a car on a hot day or in any place where the temperatures may exceed 150°F (65°C) for any extended period of time. If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.



**Bledsoe
Brace
Systems™**

by Medical Technology, Inc.

2601 Pinewood Drive
Grand Prairie, TX 75051 USA

1-888-BLEDSOE
(1-888-253-3763)

1-800-527-3666

972-647-0884

Local Fax:
972-660-5495

International Fax:
972-606-0649

www.bledsoebrace.com

Indications

Indicated for immediate application following injury or operative procedures to the elbow, or for stable or internally fixed fractures of the distal humerus or proximal to middle radius or ulna where locked or limited motion control of the elbow is desired.


Contraindications

Contraindicated for unstable fractures or for fractures of the proximal humerus or the distal radius or ulna.

Bledsoe Extender Arm Brace vs. other brands

	Bledsoe Extender Arm Brace	Other Brands
Skin Comfort	Open-cell, breathable foam laminate increases comfort and reduces possibility of skin problems.	Flame bonded technique closes foam pores and increases possibility of skin problems.
Hinge	Hinge adjusts every 15 degrees, can be adjusted without tools, and can be locked for patient security.	Hinge adjusts every 10 degrees but can be easily bumped changing the settings.
Formable Uprights	Formable uprights and plates allow proper anatomical fit to maximize comfort and safety.	Molded plastic pieces or uprights with tabs prevent forming which reduces comfort and control.
Security & Control	Straps engage and lock to brace uprights which stops the arm within the proper range for safety.	Straps pass over the uprights or through slots. Even though the hinge stops, the arm keeps moving.

Ordering Information Extender Arm Brace

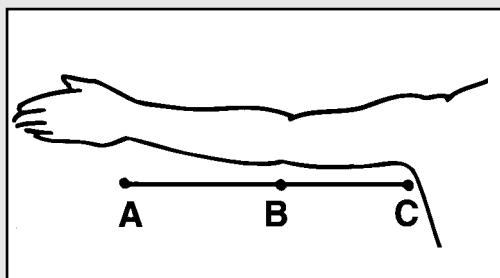

**LOWER ARM (A TO B) ULNAR STYLOID
PROCESS TO MEDIAL HUMERAL
EPICONDYLE**

Small 7" - 9 1/2" (18-24 cm)

Large 8 1/2" - 12 1/2" (21.5-32 cm)

**UPPER ARM (B TO C) MEDIAL HUMERAL
EPICONDYLE TO AXILLA**

Small 5 1/2" - 8" (14-20 cm)

Large 7" - 11 1/2" (18-29 cm)

Bledsoe Extender Arm Brace			Extender Hand Extension			
Brace Size	Left	Right	Small Left	Small Right	Large Left	Large Right
one size	AE021100	AE021200	AE012111	AE012211	AE012113	AE012213
Bledsoe Original Arm Brace			Original Hand Extension			
Brace Size	Left	Right	Left	Right		
Small	AE020101	AE020201	AE013101	AE013201		
Large	AE020103	AE020203	AE013103	AE013203		

Bledsoe limited warranty

This device is warranted to be free from defects in material and workmanship for a period of 90 days for wraps, pads, straps and normal wear components. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect.



For more detailed information on this brace including fitting and adjustment instructions, visit our Web site: www.bledsoebrace.com

For product information or questions pertaining to sales or service, please contact the sales representative in your area or Bledsoe Brace Systems.



CP010118--- Rev.G 6/04